

1 Ame'4 economy 2- 0.5mm pb eq Shields.jpg



2 Ame'4 standard 2-1mm pb eq Shields.jpg



Ame'4 Economy has 2 Gantry Blocks, & 2 Shields, each Navy Blue and 0.5mm Pb eq.

Ame'4 standard has 2 Gantry Blocks, & 2 Shields, each Black and 1mm Pb eq.

The 1mm Shields provide approximately 88% reduction in scatter Radiation to the area behind the Shields.

The 0.5mm Shields provide approximately 80% reduction in scatter Radiation to the area behind the Shields.

3 Ame'4 plus 4-0.5mm pb eq Shields.jpg

4 Ame'4 Premier 2-1 mm pb eq Gantry Shields & 2-0.5



Ame'4 Plus has 2 Gantry Blocks, 2, Door Blocks, and 4 Shields, all Navy Blue 0.5mm Pb eq.

Ame'4 Premier has 2 Gantry Blocks, 2, Door Blocks, and 2 Gantry Shields each Black 1mm Pb eq & 2 Door Shields, each Navy Blue 0.5mm Pb eq.

During transport Door Shields must be removed. Door Shields are only installed into Door Blocks once portable CT device is in position to perform exam.

5 Ame'4 Premier 2-1 mm pb eq Gantry Shields & 2 0.5mm pb eq

door open premier transport.jpg



Ame'4 Premier has 2 Gantry Blocks, 2, Door Blocks, and 2 Gantry Shields each Black 1mm Pb eq & 2 Door Shields, each Navy Blue 0.5mm Pb eq.

During transport Door Shields must be removed. Door Shields are only installed into Door Blocks once portable CT device is in position to perform exam.



Recommended storage for Door Shields – Gantry Shields may be stored in Gantry Block in the Park position.

Door Shields stored on workstation.jpg



Gantry Shields front view door closed.jpg



Gantry Shields rear view.jpg





Gantry Shields shown in the park position for transport.

Last picture clearly shows Shielding is tight against frame and does not create a driving visibility hazard.