

Ame'8 unpacking & wall mounting instructions

System weight is approximately 65 lbs / 30kg

Open the Box with the arrows pointing up (no brainer)

Inside is bubble wrap for your popping pleasure and possibly a T-shirt as our thank you!

**** carefully cut cable ties (not shielding material – again no brainer) to free shields before unloading ****

There are multiple long cable ties, used only for shipping.

Please protect table top from the stainless-steel rails with the bubble wrap, or lay them down carefully, both work fine – also the wall holder has small metal screw heads on the back of it, so use appropriate caution or lay/stand it on the floor.

Suggestion is to lay shields flat on table and then open hardware box.

There are 3 mounting holes, 2 at the top and 1 near the bottom.

3 toggle bolts are provided – you may substitute with any hardware you deem appropriate for your site.

Be sure to mount the wall holder level and high enough from the floor to keep the shields from touching the floor and allow easy access.

Suggested height is 66 inches / 1.7 meters above the floor.

Secure wall holder properly and then place shielding in position.

Place the Shield support blocks on the top shelf and add the table handgrips so everything is nice and tidy. (See picture)

